



FDA Owner Operator #10087657

USER GUIDE


See-Green® Lens Inserter (no stand)

Turn on light and attach plunger:

ALWAYS wet the bottom of the plunger with sterile saline containing no lubricant. While holding the bottom of the plunger, insert the LED bulb into the small hole with a twisting motion so only the clear part of the LED bulb enters the plunger. Don't bend wire.




Overfill your lens with saline recommended by your eye doctor to improve your chances of a bubble-free application. Let the saline drip, not stream, into the lens.

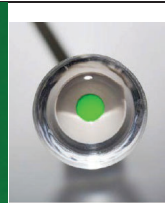

The Lid Spread is Everything.

1) Tuck your chin to your chest. Look straight down. Keep the plunger centered on your eye by focusing on the green light in the hole of the plunger. Continue approaching your eye smoothly and steadily. You will feel the fluid, but you are not done yet. Press lens onto the eye.

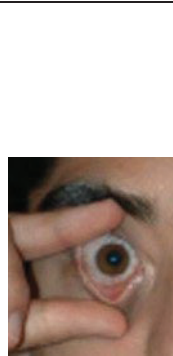

2) Capture the lens with the eyelids by releasing the lids. Don't squint. Maintain the upward pressure on the lens.

3) Release the lens from the plunger by squeezing completely. Maintain the upward pressure against the eye, while squeezing to release the lens. The lens has now been released onto your eye.



Push upward and squeeze



Stay squeezed as you pull away and retract the plunger from your eye
(courtesy BostonSight)

4) Check for bubbles. Using a mirror in a well-lit setting, check for the presence of bubbles. If bubbles are present, the lens must be removed with DMV remover and reapplied. See DMV remover instructions that came with your product.

Copyright © 2025 · Dalsey Adaptives, LLC · 1 Cottage St., Easthampton, MA 01027 U.S.A · (413) 563-2477
www.seegreenlight.com